


Geistlich
Medical

Pocket Handling Guide

Geistlich Derma-Gide®

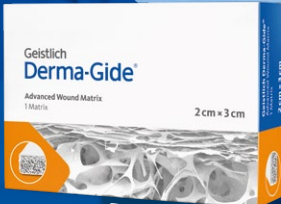

Geistlich Derma-Gide® Advanced Wound Matrix optimizes the body's own healing potential.

Excellent Handling Characteristics¹



Excellent soaking capacity. Geistlich Derma-Gide® may be applied on a freshly debrided wound.




Ready to use, with no pre-treatment, rinsing or pre-hydrating required.



Naturally conforms and adheres to the defect site as soon as hydrated.



Can be easily sutured and trimmed using standard surgical instruments.

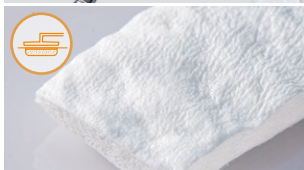

Geistlich Derma-Gide® can be applied as is when used with VAC therapy. Fenestration, if required, should be performed in a dry state.


for more extensive
information including videos
and clinical data visit
www.derma-gide.com

Convenient Clinical Application¹



Preparation of the wound bed

Prior to applying the Geistlich Derma-Gide® the affected skin site must be carefully debrided, cleaned and disinfected to ensure the wound edges contain viable tissue.

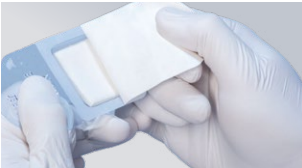

Product preparation

Open the lid after taking the inner blister out of the outer blister.



If needed, trim the device to the wound size in its dry state.



Product application

Apply Geistlich Derma-Gide® in its dry state.
Apply the product with the compact layer facing upward and the porous, soft layer toward the wound bed.



Once applied, ensure that Geistlich Derma-Gide® is completely hydrated either with wound exudate or sterile saline.

Application of secondary dressings: Geistlich Derma-Gide® should be covered with a non-adhesive dressing in order to prevent desiccation. Apply subsequent dressings in accordance with standard treatment protocols.

Ordering information

Geistlich Pharma
North America, Inc.
902 Carnegie Center
Suite 360
Princeton, NJ 08540
www.derma-gide.com

Product Information
Geistlich Derma-Gide®
Toll-free: 877-485-2968
info@geistlich-na.com

Reimbursement Hotline
Toll-free: 844-339-8148
Fax: 844-339-8149
Geistlichreimbursement@
argentaadvisors.com

How to get, how to store

Product Size	Total cm ²	Re-Order Number
12 mm Disk	1.1	500477
14 mm Disk	1.5	500478
16 mm Disk	2.0	500479
18 mm Disk	2.5	500480
1.5 cm x 2 cm	3.0	500425
2 cm x 2 cm	4.0	500474
2 cm x 3 cm	6.0	500426
2 cm x 4 cm	8.0	500475
3 cm x 4 cm	12.0	500427

Storage information: Store in a dry place at room temperature (15–25°C/59–77°F).

